# makrolux

Bedienungsanleitung  
Operating instructions  
Mode d'emploi  
Istruzioni d'uso  
Instrucciones de uso  
Gebruiksaanwijzing  
Brugsanvisning  
Bruksanvisning  
Bruksanvisning  
Käyttöohje  
Návod k obsluze  
Instrukcja obsługi  
サービスマニュアル


 **ESCHENBACH**

**makrolux**


14361

1:2,2 (2,2x)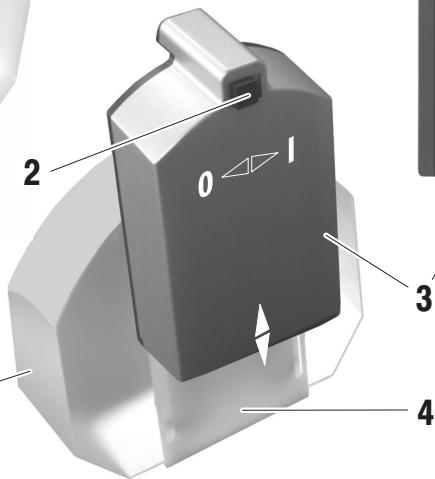

**makrolux**

14362  
1:3,6 (3,6x)

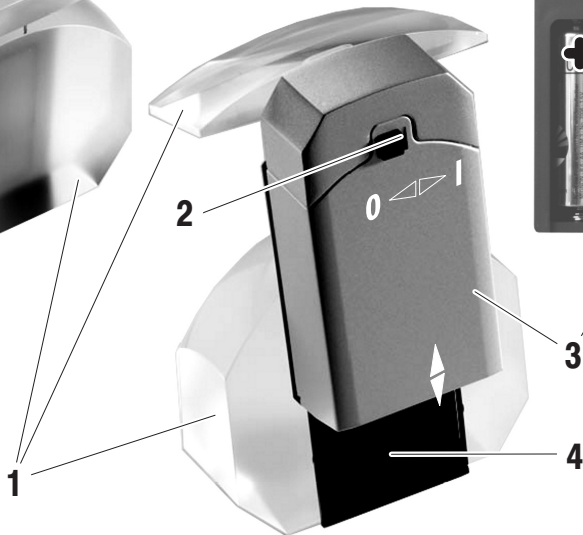<b>Deutsch</b> . . . . .	<b>4</b>
<b>English</b> . . . . .	<b>7</b>
<b>Français</b> . . . . .	<b>10</b>
<b>Italiano</b> . . . . .	<b>13</b>
<b>Español</b> . . . . .	<b>16</b>
<b>Nederlands</b> . . . . .	<b>19</b>
<b>Dansk</b> . . . . .	<b>22</b>
<b>Svenska</b> . . . . .	<b>25</b>
<b>Norsk</b> . . . . .	<b>27</b>
<b>Suomi</b> . . . . .	<b>30</b>
<b>Polski</b> . . . . .	<b>33</b>
<b>Česki</b> . . . . .	<b>36</b>
<b>日本語</b> . . . . .	<b>39</b>

## Deutsch

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## English

### Dear customer,

Congratulations on purchasing this Eschenbach bright field magnifier - a quality product "Made in Germany". We hope you enjoy using it every day.

Light and life are irrevocably linked. Senior generations need far more light than their younger counterparts. For example, to experience the same light intensity, a 60-year old requires about 15 times more light than a 10-year old. The integrated, cordless illumination by a modern, energy efficient light emitting diode (LED) is bright and nearly without wear. Your bright field magnifier features a magnification of 2.2x (3,6x) and is typified by its elegant, well-thought out shape, which offers you a comfortable reading position from an

oblique viewing angle. Its lightweight design and large viewing field, which enables reading of entire newspaper columns, are also appealing.

Do you need more magnification? As market leader for magnifying vision aids, we at Eschenbach can draw on many years of experience that are reflected in the quality, handling, and design of our products. You can obtain many other useful Eschenbach vision aids from your optician.

## Safety instructions

### ● Risk of fire!

**Lenses used in optical equipment may cause considerable damage by their burning glass effect if used or stored incorrectly. Make sure that optical lenses are never left uncovered in the sunlight.**

● **Danger of blinding or injury! Never look directly at the sun through any optical equipment! Never look directly at the light source!**

● **Protect your bright field magnifier from knocks, strikes, moisture, and excessive heat. Never place your bright field magnifier on a heating element or in direct sunlight. Use of the storage pocket supplied is recommended.**

● **Batteries must not be disposed of in household waste. Make sure that they are correctly disposed of.**

## Use

Place the bright field magnifier [1] with the even, flat side on the reading material so that the product is slightly inclined towards you. Switch on the lamp by pushing the switch [2] to the left (I).

Now adjust the distance between eye and reading material, until you see a sharp image. Use the most comfortable distance between your eye and bright field magnifier. It is dependent on the reading glasses being used and your eyes' adaptability.

Switch off the lamp when done by pushing the switch to the right (O).

## Note

If you normally wear glasses, you do not need to take them off. The bright field magnifier is designed to be used in conjunction with your ordinary reading glasses.

## Changing batteries

Switch off the lamp by pushing the switch [2] to the right (O).

Open the battery case [3] by sliding it up and taking it off the bright field magnifier.

Now take out the old batteries if necessary and put in new batteries as shown in the illustration.

● **You need 3 batteries, type Mignon (AA / LR 6) / 1.5 V.**

**Always insert three new batteries of the same type.**

**Note the position of the respective positive pole (+).**

The proper polarities are indicated in the battery case.

Close the battery case by placing it on the lid of the bright field magnifier [4] and sliding it down until it snaps tight.

## Care instructions

When the bright field magnifier is not being used, it should be kept covered or be stored in the enclosed microfibre bag.

Clean the lens with a soft, lint-free cloth (e.g. the microfibre bag or a spectacle cleaning cloth). Very dirty lenses (e.g. with fingerprints) are most effectively cleaned using lukewarm water. Do not use any soap solutions that contain softeners, alcohol-based solvents, or abrasive cleansers. These may damage the lenses.

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## Garantie

We are required by law to guarantee the function of the product described in these descriptions in respect of defects caused by manufacturing or material faults. Any damages caused by incorrect handling, including damage caused by being dropped or knocked, are not covered under the guarantee. Guarantee benefits can only be claimed through proof of purchase in the form of a sales receipt!

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Eschenbach Optik GmbH