
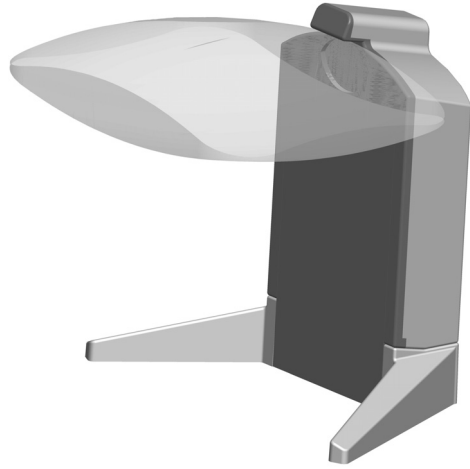
scribolux

 **ESCHENBACH**
Germany

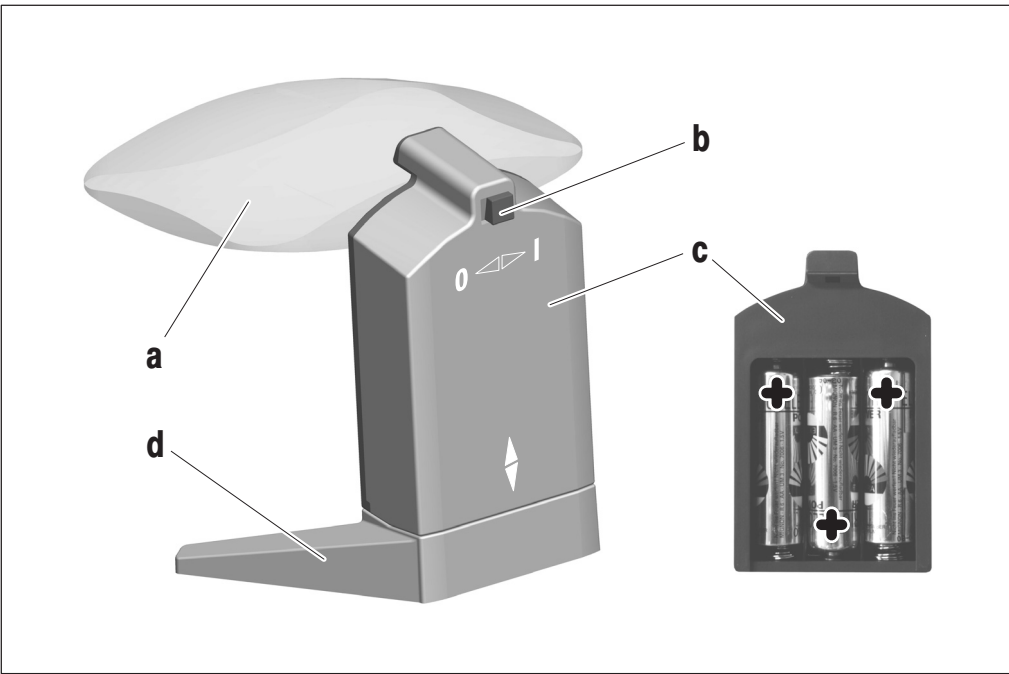Bedienungsanleitung
Operating instructions
Mode d'emploi
Istruzioni d'uso
Instrucciones de uso
Gebruiksaanwijzing
Brugsanvisning
Bruksanvisning
Bruksanvisning
Käyttöohje
Návod k obsluze
Instrukcja obsługi
サービスマニュアル

2

Deutsch	4
English	7
Français	10
Italiano	13
Español	16
Nederlands	19
Dansk	22
Svenska	25
Norsk	27
Suomi	30
Polski	33
Česky	36
日本語	39

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Dear Customer

Congratulations on purchasing this Eschenbach standing magnifier — a quality product “Made in Germany”. We hope you enjoy using it every day.

Your standing magnifier features a magnification of 2.8x. The large, open viewing field provides you with an overview of a broad area of the reading material and writing under the standing magnifier. The integrated, cordless illumination by a modern, energy-efficient light emitting diode (LED) is bright and never needs to be replaced.

Do you need more magnification? As market leader for magnifying vision aids, we at Eschenbach can draw on many years of experience that are reflected in the quality, handling, and design of our products. You can obtain many other useful Eschenbach vision aids from your optician.

Safety instructions

● Risk of fire!

Lenses used in optical equipment may cause considerable damage by their burning glass effect if used or stored incorrectly. Ensure that optical lenses are never left uncovered in the sunlight!

● Danger of blinding or injury!

Never look directly at the sun through any optical equipment!

Never look directly at the light source!

● **Protect your standing magnifier from knocks, strikes, moisture, and excessive heat. Never place your standing magnifier on a heating element or in direct sunlight. It's best if you always use the enclosed microfibre bag to protect the lens.**

● **Batteries must not be disposed of in household waste. Ensure that they are correctly disposed of.**

Use

Place the standing magnifier on the reading material so that the lens [a] points towards you.

Switch on the lamp by pushing the switch [b] to the left (I).

Use the most comfortable distance between your eye and standing magnifier. It is dependent on the reading glasses being used and your eyes' adaptability.

Switch off the lamp when done by pushing the switch to the right (O).

Changing batteries

Switch off the lamp by pushing the switch [b] to the right (O).

Open the battery compartment [c] by pulling it off upwards from the standing magnifier.

Now take out the old batteries if necessary and put in new batteries as shown in the illustration.

● You need 3 batteries, type Mignon

(AA / LR 6) / 1.5 V.

Always insert three new batteries of the same type.

Note the position of the respective positive pole (+).

The proper polarities are indicated in the battery case.

Close the battery case by placing it on the standing magnifier [d] and sliding it down until it snaps tight.

Note

If you normally wear glasses, you do not need to take them off. The standing magnifier is designed to be used also in conjunction with your ordinary reading glasses.

Care instructions

As long as the standing magnifier is not being used, the enclosed microfibre bag should be pulled over the lens.

● **Do not use any soap solutions that contain softeners, alcohol-based solvents, or abrasive cleansers.**

● **Do not rinse the standing magnifier under running water!**

● **Do not clean the standing magnifier in an ultrasonic bath!**

Clean the lens with a soft, lint-free cloth (e.g. with the microfibre bag or with a spectacle cleaning cloth). Moisten the cleaning cloth to remove more persistent stains (e.g. finger prints).

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Garantie

We are required by law to guarantee the function of the product described in these descriptions in respect of defects caused by manufacturing or material faults. Any damages caused by incorrect handling, including damage caused by being dropped or knocked, are not covered under the guarantee. Guarantee benefits can only be claimed through proof of purchase in the form of a sales receipt!

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Eschenbach Optik GmbH